Instructions for Use

MRX PT Quick (INR) Calibrator Set, K5043

Contents

<u>English [EN]</u>	<u>3</u>
<u>Español [ES]</u>	<u>6</u>
<u>Italiano [IT]</u>	<u>9</u>

Instructions for Use [EN]

MRX PT Quick (INR) Calibrator Set

REF K5043

For *In vitro* Diagnostic Use.

1 Intended use

For calibration of the MRX PT Quick reagent to determine the local International Sensitivity Index (ISI) and the Normal Clotting Time (NCT). Intended to be used by professional laboratory personnel using coagulation analysers.

2 Background and principle of method

For background information and principle of the method, refer to the Instructions for Use associated with MRX PT Quick (K5024).

3 Components

MRX PT Quick (INR) Calibrator Set consists of six vials with lyophilised citrated human plasma with INR 0.9 - 6.0.

Each set contains:

Name	No of vials	Volume
PT Quick (INR) Calibrator 1	1	1 mL
PT Quick (INR) Calibrator 2	1	1 mL
PT Quick (INR) Calibrator 3	1	1 mL
PT Quick (INR) Calibrator 4	1	1 mL
PT Quick (INR) Calibrator 5	1	1 mL
PT Quick (INR) Calibrator 6	1	1 mL

4 Metrological traceability

Each new lot of MRX PT Quick (INR) Calibrator Set is assigned with MRX PT Quick against in-house reference material with traceability to WHO IRP 67/40 (via RBT/05) determined by the WHO reference procedure (manual tilt tube technique). Refer to the Certificate of Analysis or the label for the lot-specific INR values.

Specification:

Calibrator	Specification
PT Quick (INR) Calibrator 1	0.9 - 1.2
PT Quick (INR) Calibrator 2	> Calibrator 1
PT Quick (INR) Calibrator 3	> Calibrator 2
PT Quick (INR) Calibrator 4	> Calibrator 3
PT Quick (INR) Calibrator 5	> Calibrator 4
PT Quick (INR) Calibrator 6	> Calibrator 5

5 Warnings and precautions

Wear suitable clothing for protection. Avoid contact with skin and eyes. Do not empty into drains. Waste must be disposed of in accordance with local regulations.

The calibrators contain material of human origin. Each donor has been tested by approved methods and found negative for the presence of HBsAg and anti-HIV I & II and anti-HCV. However, as no method can offer complete assurance that infectious agents are absent, this material should be handled as potentially infectious.

The calibrators (3-6) contain Bovine Serum Albumin. The animals were approved by veterinarians by ante- and post-mortem inspections. However, as no method can offer complete assurance, this material should be handled as potentially infectious.

6 Preparation

- Allow the vials to equilibrate at 15 - 25 °C for at least 10 minutes before reconstitution.
- Before opening, carefully tap the vials containing Calibrator 1 - 6 against a surface to collect the lyophilised material at the bottom.
- Add 1.00 mL deionised water (e.g. MRX Laboratory Water, K5036) to each vial. The water temperature should be 15 - 25 °C.
- Reseal the vials and swirl gently. Keep reconstituted calibrators at 15 - 25 °C for at least 15 minutes before use.

- Make sure that the content is completely reconstituted and mix thoroughly by swirling before use.

7 Storage and stability

Store at 2 - 8 °C. After reconstitution, stable for 8 hours at 15 - 25 °C in the closed original vial, provided no contamination occurs.

8 Specimen collection and preparation

For specimen collection and preparation, refer to the Instructions for Use associated with MRX PT Quick (K5024).

9 Procedure

For each instrument, refer to its operator's manual and to the instrument-specific application sheet.

For calculation of local ISI and NCT, please download *Calculation Form for MRX PT Quick (INR) Calibrator Set* on the homepage of Nordic Biomarker:

<https://www.nordicbiomarker.com/cal-pt-quick>

1. Prepare the PT Quick reagent (e.g. MRX PT Quick) to be calibrated according to instructions for use for the specific PT Quick reagent.
2. Reconstitute PT Quick (INR) Calibrator 1 - 6 according to section 6.
3. Measure the clotting time in seconds for each calibrator in duplicates.
4. Insert the clotting time for each measurement in the Calculation Form for MRX PT Quick (INR) Calibrator Set.
5. Insert the corresponding INR value for each calibrator in the column labelled INR in the Calculation Form for MRX PT Quick (INR) Calibrator Set. Lot specific INR for each calibrator is specified on the vial label and in the certificate of analysis for MRX PT Quick (INR) Calibrator Set.
6. When step 4 and 5 are completed, the reagent and instrument specific ISI and NCT values are shown under Results.

10 Material required but not provided

Coagulation analyser, pipettes, and the following:

Reagent	REF
MRX PT Quick	K5024

Control material	REF
MRX Routine Normal Control	K5039
MRX Routine Abnormal Control	K5040

Solutions	REF
Deionised water for reconstitution e.g. MRX Laboratory Water	K5036

11 Quality control

To maintain consistent assay results, it is recommended that control plasmas are assayed at regular intervals. MRX Routine Controls (K5039/K5040) are recommended for MRX PT Quick. Each laboratory should establish a control range to determine the allowable variation in the day-to-day performance of the test, as well as appropriate intervals for analysing controls in accordance with good laboratory practice.

12 Results

Use of MRX PT Quick (INR) Calibrator Set generates ISI and NCT for reporting results obtained using MRX PT Quick in International Normalised Ratio, INR.

13 Expected values

For expected values, refer to the Instructions for Use associated with MRX PT Quick (K5024).

14 Limitations and interfering substances

For limitations and interfering substances, refer to the Instructions for Use associated with MRX PT Quick (K5024).

15 Analytical performance characteristics

For analytical performance characteristics, refer to the Instructions for Use associated with MRX PT Quick (K5024).

16 Reporting of incidents

Any serious incidents that occur in relation to this device shall be reported to Nordic Biomarker as well as the national competent authority in which the user is established.

17 Additional information

A paper copy of these Instructions for Use is available on request. Contact your local distributor.

The instrument-specific application sheet is available from your local distributor.

18 Definition of symbols

Manufacturer

Use-by date

CE mark

Temperature limit

In vitro diagnostic medical device

Biological risks

Catalogue number

Contains biological material of animal origin

Batch code

Contains human blood or plasma derivatives

Consult electronic instructions for use

nordicbiomarker.com/IFU**19 Revision history**

Version	Changes to previous version
4.0	Added translation into Italian.

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